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**Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!**

## The test kit contains:

- Instruction manual for fecal sample collection
- Stool collection tube (brown cap)
- Feces catcher
- RigidSafetyBag®

E

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon-Fri 8:30-12:30 and 13:30-17:00) or by e-mail: [info@rpsh.nl](mailto:info@rpsh.nl).

## Preparations

1. Fill in your name and date of birth on the label of the stool collection tube.
2. Check if your personal details have been filled in correctly on the application form.
3. Fill in the date of the sample collection on the application form on each collection day, and agree to the terms and conditions.
4. Watch the following instructional video by scanning the QR-code:

Feces

Or watch via the website:  
[www.rpsh.nl/rpsh-analytic/testkits/feces/](http://www.rpsh.nl/rpsh-analytic/testkits/feces/)

## Important!

- With flush toilets, the stool falls directly into the water. This makes collection difficult and can reduce the reliability of the test. In addition to the feces catcher, it is recommended to place a few longer pieces of toilet paper folded in half on the surface of the water. The stool can then be collected from the feces catcher or the toilet paper.

## Additional guidelines for women

Please do not collect stool during menstruation.

## Sample collection

1. Attach the feces catcher to the toilet seat and collect your stool from it.
2. Fill the stool collection tube at least halfway with stool.  
**Please note:** Take a scoop of stool from at least 6 different places.
3. Close the stool collection tube and place it in the RigidSafetyBag® in the hard plastic cover.
4. Press the exposed adhesive strip so that the bag is properly sealed.  
– *The RigidSafetyBag® contains absorbent material that protects the collection tube during transport.*
5. Please check the sample collection date on the application form and whether you have agreed to the terms and conditions.
6. Place the RigidSafetyBag® together with the application form in the return envelope and seal it. The return envelope is now ready to be send to our laboratory.

## Shipping

**NL: RPSH Analytic**, antwoordnummer 1461, 4460 VS Kamperland

**Belgium: RPSH Analytic**, drop off at an UPS Access Point

**Germany: RPSH Analytic**, Postfach 1171, 50140 Kerpen, Deutschland

**Other countries:** please contact us at [info@rpsn.nl](mailto:info@rpsn.nl)

**Please note: Do not send during the weekend or the day before a national holiday!**

**NL:** Within the Netherlands, send from **Monday to Thursday** (from an orange PostNL postbox).

1. Check whether the postbox has already been emptied.
2. Use a medical postbox if the regular postbox is already empty. These postboxes can be recognised by the logo 'medische post'.
3. Find a medical postbox via the PostNL Location Finder, filter on 'post a letter' and select 'medical' at the bottom.

Scan the QR code for the locations of the medical postboxes:

<https://www.postnl.nl/en/locationfinder/>

**MEDISCHE  
POST**

**Please note:** If the test kit is not sent via a medical postbox, but via a PostNL point, it will be treated as a parcel. This may delay delivery and may prevent the test kit from being processed due to the quality of the sample.

**Belgium:** From Belgium send from **Monday to Wednesday** via UPS.

- After taking the test kit, the patient can drop the sealed return envelope off at one of the UPS Access Points (do NOT deposit it in a Bpost mailbox!)
- Send from Monday to Wednesday (this guarantees that the test kit can be processed before the weekend)
- The cost for the return shipment is paid by RPSH.

The UPS Access Points can be found on the UPS website under 'Locations' at the top right [[ups.com/dropoff/?loc=nl\\_BE](https://ups.com/dropoff/?loc=nl_BE)]

**Please note:** The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!