TRP/HIS-testkit

RPSH
Sanitas Humanus
● ● ●

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Please read the instructions completely before you start taking the samples! If you have any questions: please contact your practitioner first!

The test kit contains:

- Instruction manual for fecal sample collection
- TRP/HIS tube (yellow/blue cap with stabilizer)

Please note: The stabilizer is a corrosive liquid, do not bring into contact with skin. Keep out of reach of children. **Do not pour the stabilizer out of the collection tube.**

E

- Label for TRP/HIS tube
- Feces catcher
- RigidSafetyBag®

You also received a return envelope and application form from RPSH Analytic. Is the test kit incomplete? Please contact us by phone at +31(0)113-370170 (Mon–Fri 8:30–12:30 and 13:30–17:00) or by e-mail: info@rpsh.nl.

Preparations

1. Fill in your name and date of birth on the label of the TRP/HIS tube and stick it on the TRP/HIS tube.
2. Check if your personal details have been filled in correctly on the application form.
3. Check if there is liquid in the TRP/HIS tube. If not, please contact us first on the aforementioned details.
4. Fill in the date of the sample collection on the application form on each collection day, and agree to the terms and conditions.
5. Watch the following instructional video by scanning the QR-code:

Or watch via the website:
www.rpsh.nl/rpsh-analytic/testkits/histamine/


- With flush toilets, the stool falls directly into the water. This makes collection difficult and can reduce the reliability of the test. In addition to the feces catcher, it is recommended to place a few longer pieces of toilet paper folded in half on the surface of the water. The stool can then be collected from the feces catcher or the toilet paper.

Please do not collect stool during menstruation.

1. Attach the feces catcher to the toilet seat and collect your stool from it.


- Please note:** the liquid must remain in the tube!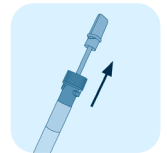

- Make sure that only the notches of the stick are filled.*


Too much sample material leads to a false-positive test result!

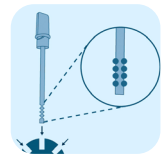- 
- A diagram showing a syringe with a needle attached. A black arrow points downwards, indicating the direction to pull the needle away from the syringe.

5. Close the collection tube tightly.

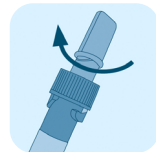6. Place the TRP/HIS tube in the hard plastic case.

7. Remove the white foil from the RigidSafetyBag®. Press the exposed adhesive strip so that the bag is properly sealed.
– The RigidSafetyBag® contains absorbent material that protects the tube during transport.

8. Please check the sample collection date on the application form and whether you have agreed to the terms and conditions.

9. Place the Safetybag together with the application form in the return envelope and seal it. The return envelope is now ready to be send to our laboratory.

Please note: If both Tryptophan and Histamine are requested, two separate collection tubes must be submitted. Both tests (biomarkers) cannot be conducted from a single tube.

Other countries: please contact us at info@rps.nl

Please note: The UPS Access Point you select will indicate the latest collection time for 'Standard' service. The test kit must be dropped off before this time to be delivered to RPSH Analytic the next day!